


DESIGN

Veronique®
c o m p r e s s i o n w e a r

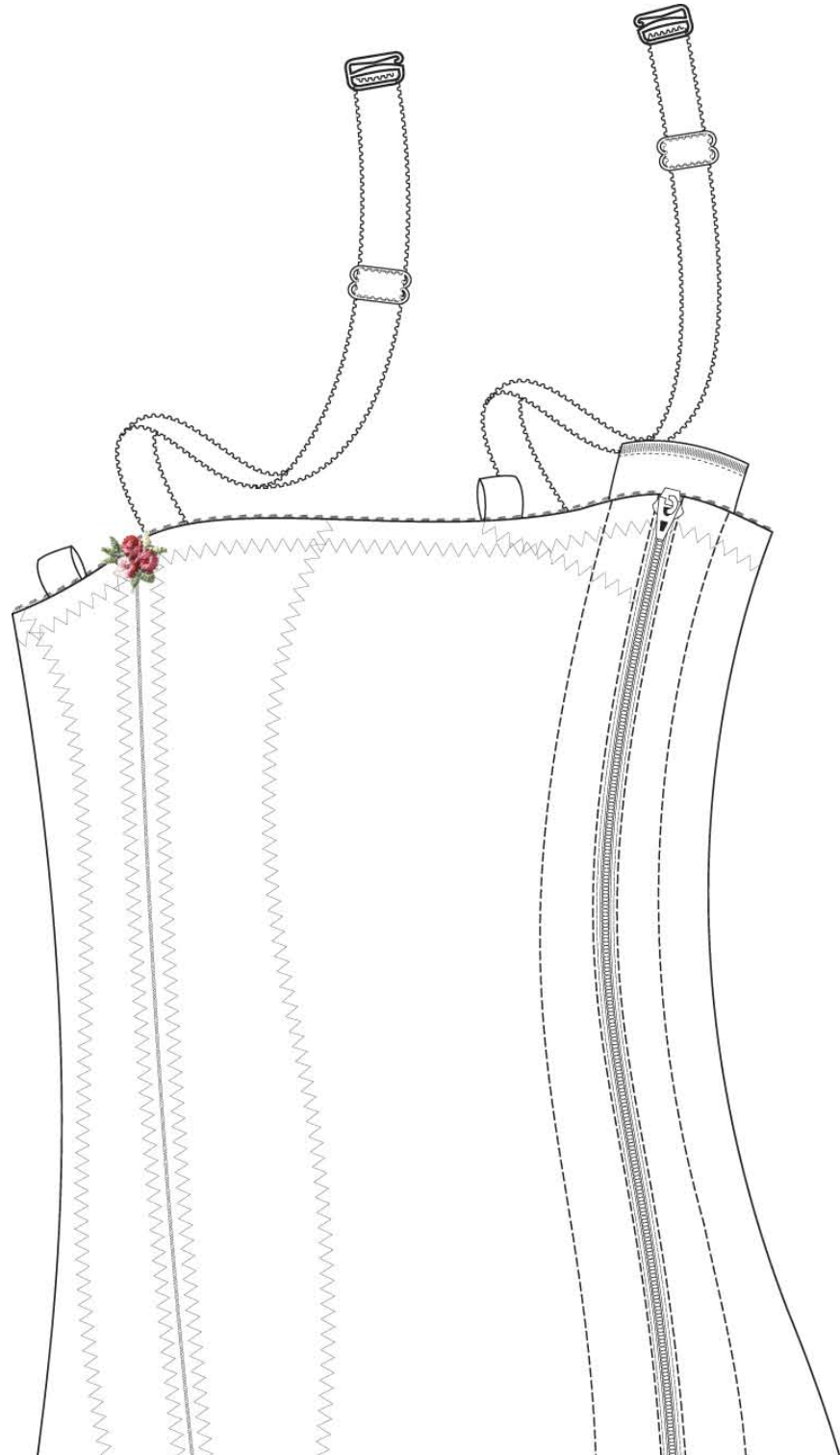


Body Contouring Garments

Version 08.03.1



Body Contouring Garments

Design Veronique® body contouring garments are designed to provide the ideal level of targeted compression after any body contouring procedure- from liposuction to abdominoplasty. Our collection of body contouring garments features three garment series: **Mid-Abdominal Girdles** with a wide waistband, **Full Abdominal Girdles** with adjustable shoulder straps and **Full Abdominal Girdles** with a seamless bra. All are available in zippered and non-zippered styles and a size range of XS-4X. Each and every garment is manufactured to our exacting quality standards in our California corporate facility.

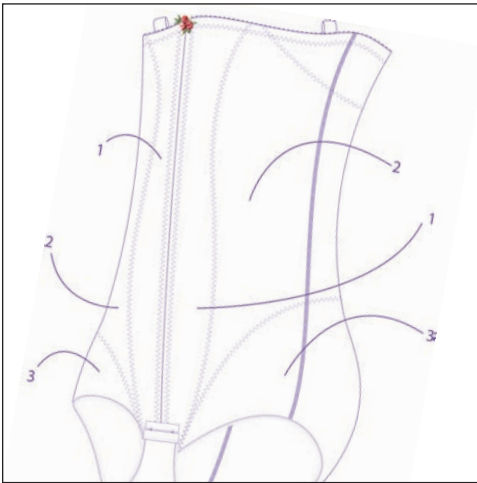

Every Design Veronique® garment incorporates gradient compression technology and strategically placed inner panels in key surgical areas for the right compression in the right places. This precision design technology delivers targeted compression for key surgical areas, better overall comfort and enhances healing.

Our designers use Computer Assisted Design technology to ensure the accuracy of fit and compression provided in each part of the garment. Every design is tested to ensure proper fit, level of compression and durability.

Compression Panels

Wide Size Range

Comfortable Fit

Gradient Compression Technology

Latex Free

Powernet Fabric

Over 85 Styles

Made in the USA

Zippered & Non-Zip Styles

Innovative Designs

Mid-Abdominal Girdles with Waistband

Provide ideal compression for procedures of the lower abdomen, hip, thigh and knee. They are constructed of a breathable powernet fabric and feature a wide waistband, targeted inner panels and gripper lace on leg. They are available in zippered & non-zippered styles.

Sizes: XS - 4X
Colors: Black, White, Champagne



#75
**Zippered
Abdominal Brief**

Ideal for:
Lower abdominal
procedures



#145
Non-Zippered Abdominal Brief



Full Abdominal Girdles with Adjustable Straps

Designed for full abdominal, hip, thigh and knee procedures. Constructed of a breathable powernet fabric they feature adjustable detachable suspenders, targeted inner panels and gripper lace on leg. Available in zippered & non-zippered styles.

Sizes: XS - 4X
Colors: Black, White, Champagne



#85
**Zippered
Torso Brief**

Ideal for:
Abdominal procedures



#165
Non-Zippered Torso Brief



Full Abdominal Girdles with Seamless Bra

Ideal for multiple point Liposuction, body contouring and procedures of the abdomen, back, breast, hip, thigh and knee. They are constructed of a breathable powernet fabric and feature built-in seamless cotton knit bra, wide shoulder straps, targeted inner panels and gripper lace on leg. They are available in zippered & non-zippered styles (B850-TCP is only available in zippered style).

Sizes: XS - 4X
Colors: Black, White, Champagne



#B85
**Zippered High-Back
Torso Brief with Bra**

Ideal for:
Abdominal, back, and breast
procedures



#B165
**Non-Zippered High-Back
Torso Brief with Bra**



To place an order or request a full catalog call:

#750
Zippered
Tram Flap Girdle

Ideal for:
 Tram flap procedures



#1450
 Non-Zippered Tram Flap Girdle



#753
Zippered
Above-Knee Girdle

Ideal for:
 Lower abdominal, hip, and thigh procedures




#1453
 Non-Zippered Above-Knee Girdle



#755
Zippered
Below-Knee Girdle

Ideal for:
 Lower abdominal, hip, thigh, and knee procedures

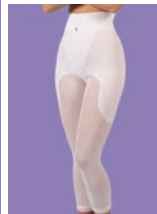


#1455
 Non-Zippered Below-Knee Girdle

#850
Zippered
Abdominal Girdle

Ideal for:
 Abdominal procedures



#1650
 Non-Zippered Abdominal Girdle

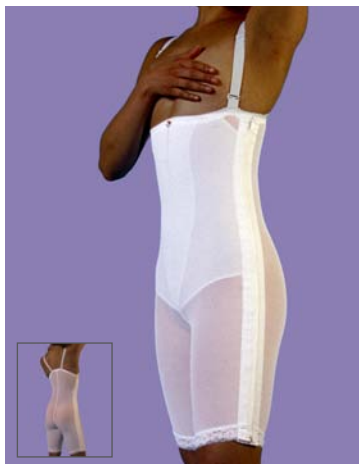

#853
Zippered
Body Girdle

Ideal for:
 Abdominal, hip, and thigh procedures




#1653
 Non-Zippered Body Girdle



#855
Zippered
Full-Body Girdle

Ideal for:
 Abdominal, hip, thigh, and knee procedures

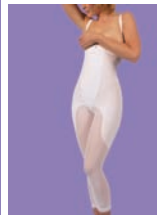

#1655
 Non-Zippered Full-Body Girdle

#B850-TCP
Zippered
High-Back Abdominal
Girdle with Bra

Ideal for:
 Abdominal, flank, back, and breast procedures that require targeted compression immediately post-op
(only available in zippered style)



#B853
Zippered
Above-Knee High-Back
Girdle with Bra

Ideal for:
 Abdominal, hip, thigh, back and breast procedures




#B1653
 Non-Zippered Above-Knee High-Back Girdle with Bra



#B855
Zippered
Below-Knee High-Back
Girdle with Bra

Ideal for:
 Abdominal, hip, thigh, back, knee, and breast procedures

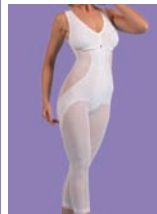

#B1655
 Non-Zippered Below-Knee High-Back Girdle with Bra

Design Veronique® Body Contouring Girdles size is based upon waist and lower hip measurements.

Keep in mind that compression garments are designed for a snug fit. Optimal compression and support require accurate measurements. If you desire a looser fit to accommodate dressings or excessive swelling, we suggest ordering one size larger than the size suggested by your measurements.

MEASURING INSTRUCTIONS:

To ensure a proper fit, patients should avoid wearing bulky clothing during measuring. Feet should be 2-3" apart and arms should be relaxed at their side. We recommend using a cloth measuring tape; hold the tape level and firmly, but do not allow it to "dig" into their skin or clothing.

Waist: Measure around the narrowest part of the waistline and above the navel.


Lower Hip Area: Measure around the fullest part of the lower hip area, keep tape measure parallel to the floor.

To find the appropriate size, select the size in which your waist and hip measurements match.

Measuring and fit tips...

- Measure twice for accuracy
- Remove shoes
- When between sizes; choose larger size
- Bicep applies to styles with arms only
- Under bust measurement not relevant for girdles
- Soft cotton knit allows the bra to fit a wide range of cups sizes
- The bra size matches the garment size
- Bra cups are cut on the bias for a superior contour

Sizing

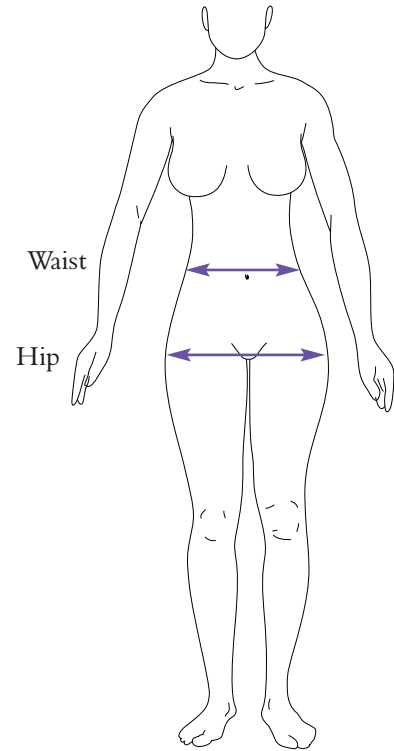

To find the appropriate size, match your waist and hip measurements to the chart below. If measurements fall into different sizes choose the larger size. *i.e. hips are Medium (37") & waist is Large (32"), choose size Large*

Size	Waist	Hip
XS	21" - 23"	31" - 33"
S	24" - 26"	34" - 36"
M	27" - 29"	37" - 39"
L	30" - 32"	40" - 42"
XL	33" - 35"	43" - 45"
XXL	36" - 39"	45" - 48"
3X	40" - 43"	49" - 52"
4X	44" - 47"	53" - 56"



*To view other product catalogs visit us online at:
www.DesignVeronique.com/productcatalogs*



*999 Marina Way South, Richmond CA 94804
800.442.5800 • 510.970.7990 • 510.970.7996 fax
www.DesignVeronique.com
email: cs@designveronique.com*