

# Tips on Choosing the Appropriate Implantech® Chin Implant

## Anterior Projection-Chart 2

| Implant                           | Size    | Catalog Number | Anterior Projection |
|-----------------------------------|---------|----------------|---------------------|
| Terino Extended Anatomical™ Chin  | Small   | TEAC-S         | 0.38                |
| Conform™ Extended Anatomical Chin | Small   | CEAC-S         | 0.43                |
| Extended Anatomical Chin          | Small   | EAC-S          | 0.45                |
| Curvilinear Silicone Chin         | X-Small | CSC-XS         | 0.45                |
| Terino Extended Anatomical™ Chin  | Medium  | TEAC-M         | 0.46                |
| Conform™ Extended Anatomical Chin | Medium  | CEAC-M         | 0.50                |
| Extended Anatomical Chin          | Medium  | EAC-M          | 0.50                |
| Flowers Mandibular Glove®         | X-Small | FMG-VXS        | 0.50                |
| Terino Extended Anatomical™ Chin  | Large   | TEAC-L         | 0.58                |
| Conform™ Extended Anatomical Chin | Large   | CEAC-L         | 0.60                |
| Extended Anatomical Chin          | Large   | EAC-L          | 0.60                |
| Flowers Mandibular Glove®         | Small   | FMG-VS         | 0.60                |
| Anatomical Chin                   | Small   | AC-S           | 0.60                |
| Terino Extended Anatomical™ Chin  | X-Large | TEAC-XL        | 0.64                |
| Mittelman Pre Jowl Chin™          | Small   | MPJC-S         | 0.65                |
| Curvilinear Silicone Chin         | Small   | CSC-S          | 0.65                |
| Curvilinear Silicone Chin         | Medium  | CSC-M          | 0.65                |
| Curvilinear Silicone Chin         | Large   | CSC-L          | 0.65                |
| Curvilinear Silicone Chin         | X-Large | CSC-XL         | 0.70                |
| Conform™ Extended Anatomical Chin | X-Large | CEAC-XL        | 0.75                |
| Extended Anatomical Chin          | X-Large | EAC-XL         | 0.75                |
| Vertical Lengthening Chin         | Small   | VLC-S          | 0.79                |
| Flowers Mandibular Glove®         | Medium  | FMG-VM         | 0.80                |
| Mittelman Pre Jowl Chin™          | Medium  | MPJC-M         | 0.80                |
| Anatomical Chin                   | Medium  | AC-M           | 0.90                |
| Mittelman Pre Jowl Chin™          | Large   | MPJC-L         | 0.95                |
| Vertical Lengthening Chin         | Medium  | VLC-M          | 1.00                |
| Flowers Mandibular Glove®         | Large   | FMG-VL         | 1.00                |
| Anatomical Chin                   | Large   | AC-L           | 1.00                |
| Mittelman Pre Jowl Chin™          | X-Large | MPJC-XL        | 1.10                |
| Vertical Lengthening Chin         | Large   | VLC-L          | 1.20                |
| Flowers Mandibular Glove®         | X-Large | FMG-VXL        | 1.20                |
| Anatomical Chin                   | X-Large | AC-XL          | 1.20                |

Dimensions in centimeters

**Suggested as a sizing guide by Steven J. Pearlman, MD, FACS**



CIMEOSIL® | ALLIED BIOMEDICAL® | GELZONE®

800.733.0833 | implantech.com

# Tips on Choosing the Appropriate Implantech® Chin Implant

## Desired Outcome

| Anterior Projection-Chart 1       | Catalog Number | X-Small   | Small | Medium | Large | X-Large |
|-----------------------------------|----------------|---|-------|--------|-------|---------|
| Terino Extended Anatomical™ Chin  | TEAC           |   | 0.38  | 0.46   | 0.58  | 0.64    |
| Conform™ Extended Anatomical Chin | CEAC           |   | 0.43  | 0.50   | 0.60  | 0.75    |
| Extended Anatomical Chin          | EAC            |   | 0.45  | 0.50   | 0.60  | 0.75    |
| Curvilinear Chin (button)         | CSC            | 0.45  | 0.65  | 0.65   | 0.65  | 0.70    |
| Flowers Mandibular Glove®         | FMG            | 0.50  | 0.60  | 0.80   | 1.00  | 1.20    |
| Anatomical Chin                   | AC             |   | 0.60  | 0.90   | 1.00  | 1.20    |
| Mittelman Pre Jowl Chin™          | MPJC           |   | 0.65  | 0.80   | 0.95  | 1.10    |
| Vertical Lengthening Chin         | VLC            |   | 0.79  | 1.00   | 1.20  |         |
| Glasgold Wafer™                   | GLE-2/GLM-2    | Provides additional .20 cm to MPJC, EAC, and TEAC |       |        |       |         |

*Dimensions in centimeters*

## Anterior Projection-Chart 2 (on reverse)

| Lateral Projection                              | Catalog Number | X-Small | Small | Medium | Large | X-Large |
|---|----------------|---------|-------|--------|-------|---------|
| Conform™ Extended Anatomical Chin               | CEAC           |         | 0.20  | 0.30   | 0.30  | 0.30    |
| Flowers Mandibular Glove®                       | FMG            | 0.20    | 0.30  | 0.30   | 0.40  | 0.50    |
| Extended Anatomical Chin                        | EAC            |         | 0.20  | 0.40   | 0.40  | 0.50    |
| Terino Extended Anatomical™ Chin                | TEAC           |         | 0.30  | 0.30   | 0.30  | 0.40    |
| Mittelman Pre Jowl®<br>(no anterior projection) | MPJ            |         | 0.30  | 0.40   | 0.50  | 0.60    |
| Mittelman Pre Jowl Chin™                        | MPJC           |         | 0.33  | 0.40   | 0.50  | 0.56    |
| Vertical Lengthening Chin                       | VLC            |         | 0.52  | 0.50   | 0.60  |         |

*Dimensions in centimeters*

| Vertical Height                   | Catalog Number | X-Small | Small | Medium | Large | X-Large |
|-----------------------------------|----------------|---------|-------|--------|-------|---------|
| Terino Extended Anatomical™ Chin  | TEAC           |         | 1.00  | 1.20   | 1.30  | 1.40    |
| Extended Anatomical Chin          | EAC            |         | 1.00  | 1.20   | 1.40  | 1.40    |
| Conform™ Extended Anatomical Chin | CEAC           |         | 1.00  | 1.20   | 1.40  | 1.50    |
| Curvilinear Chin (button)         | CSC            | 1.10    | 1.40  | 1.40   | 1.40  | 1.50    |
| Anatomical Chin                   | AC             |         | 1.20  | 1.20   | 1.20  | 1.20    |
| Vertical Lengthening Chin         | VLC            |         | 1.40  | 1.60   | 1.80  |         |
| Mittelman Pre Jowl Chin™          | MPJC           |         | 1.50  | 1.50   | 1.50  | 1.50    |
| Flowers Mandibular Glove®         | FMG            | 1.75    | 1.75  | 1.75   | 1.75  | 1.75    |

*Dimensions in centimeters*

## Other Desired Characteristics

### Square Appearance

Terino Extended Anatomical™ Chin  
Terino Square Chin-Style I  
Terino Square Chin-Style II

### Slight tilt upwards

Flowers Mandibular Glove®

### Vertical Dimension

Vertical Lengthening Chin

### Stability

Implants with holes for integration:

Curvilinear Chin (button)  
Mittelman Pre Jowl®  
Mittelman Pre Jowl Chin™

Conforming (more flexible):

Conform™ Extended Anatomical Chin

Allows Tissue Attachment:

ePTFE-Coated Extended Anatomical Chin