

Cotswold Microdissection Needles



- Ultra sharp polished tip for precise soft tissue dissection.
- Tungsten electrode maintains sharpness throughout procedure
- Highly effective at low power settings
- Minimum eschar build up - easy to clean
- Single use product

The most innovative range
of microdissection needles



Cotswold microdissection needles


Cotswold microdissection needles offer improved cutting and coagulation performance, whilst minimising thermal spread, offering the surgeon the precision required for exceptionally fine dissection in a wide variety of procedures.

Precise electrosurgery

The Cotswold needle tungsten tip remains sharp throughout the procedure. With a melting point greater than 3420° C, the needle will not become blunt, especially at the low power settings.

The effective alternative

The fine tungsten tip of the Cotswold needle allows for greater accuracy to be achieved at very low power settings, in both cut and coag modes. This accuracy cannot be achieved with traditional stainless steel needles. Due to the extremely small surface area, a focused spark plasma is created at very low power settings, almost eliminating tissue drag and thermal damage, which could cause scarring.

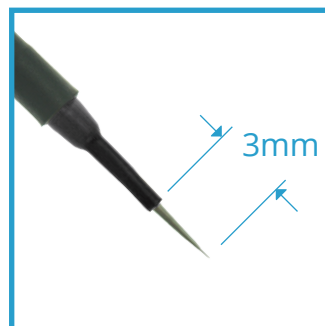

Safe - unique fingergrip allows for easy insertion and extraction, without limiting vision and prevents needlestick injuries

Straight Needles

PMS1650 - Cotswold Needle 30mm shaft, 10mm straight needle, pack of 10



PMS1651 - Cotswold Needle 30mm shaft, 20mm straight needle, pack of 10



PMS1614 - Cotswold Needle 40mm shaft, 20mm straight needle, pack of 10



PMS1654 - Cotswold Needle 55mm shaft, 10mm straight needle, pack of 10



PMS1602 - Cotswold Needle 65mm shaft, 10mm straight needle, pack of 10



PMS1656 - Cotswold Needle 75mm shaft, 10mm straight needle, pack of 10



PMS1613 - Cotswold Needle 100mm shaft, 10mm straight needle, pack of 10



PMS1612 - Cotswold Needle 130mm shaft, 10mm straight needle, pack of 5




PMS1611 - Cotswold Needle 150mm shaft, 10mm straight needle, pack of 5



PMS1610 - Cotswold Needle 180mm shaft, 10mm straight needle, pack of 5




Cotswold Microdissection Needles

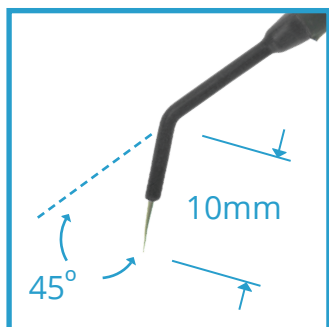

45° Angled Needles 10mm Tip

PMS1653 - Cotswold Needle 30mm shaft, 10mm 45° angled needle, pack of 10

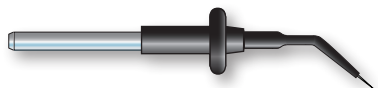


PMS1631 - Cotswold Needle 40mm shaft, 10mm 45° angled needle, pack of 10



PMS1655 - Cotswold Needle 55mm shaft, 10mm 45° angled needle, pack of 10




PMS1603 - Cotswold Needle 65mm shaft, 10mm 45° angled needle, pack of 10

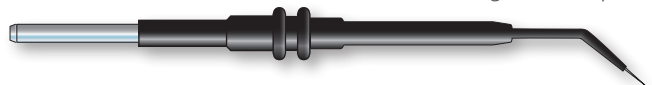

PMS1605 - Cotswold Needle 75mm shaft, 10mm 45° angled needle, pack of 10



PMS1623 - Cotswold Needle 100mm shaft, 10mm 45° angled needle, pack of 10

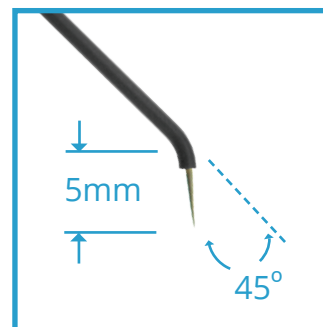


45° Angled Needles 5mm Tip

PMS1692 - Cotswold Needle 30mm shaft, 5mm 45° angled needle, pack of 10



PMS1693 - Cotswold Needle 40mm shaft, 5mm 45° angled needle, pack of 10

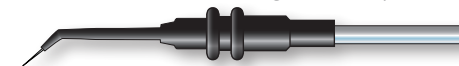

PMS1694 - Cotswold Needle 55mm shaft, 5mm 45° angled needle, pack of 10



PMS1695 - Cotswold Needle 75mm shaft, 5mm 45° angled needle, pack of 10



PMS1696 - Cotswold Needle 100mm shaft, 5mm 45° angled needle, pack of 10




PMS1697 - Cotswold Needle 130mm shaft, 5mm 45° angled needle, pack of 10




90° Angled Needles



PMS1660 - Cotswold Needle 30mm shaft, 10mm 90° angled needle, pack of 10

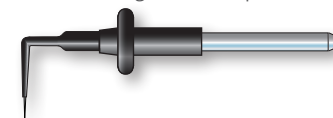

PMS1661 - Cotswold Needle 40mm shaft, 10mm 90° angled needle, pack of 10

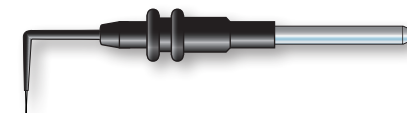

PMS1662 - Cotswold Needle 55mm shaft, 10mm 90° angled needle, pack of 10

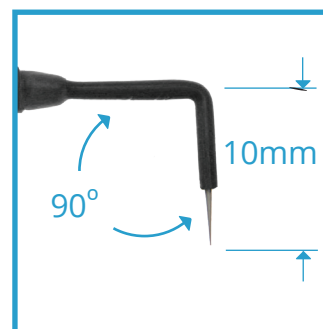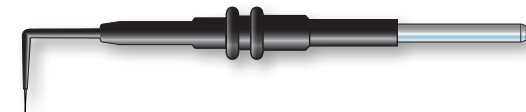

Cotswold needles can be used in the following procedures...



Cotswold needle product codes														
PMS1696	PMS1695	PMS1693	PMS1692	PMS1656	PMS1654	PMS1653	PMS1651	PMS1650	PMS1614	PMS1613	PMS1612	PMS1605		
				✓	✓		✓		✓				Abdominoplasty	Aesthetic procedures
		✓			✓	✓	✓		✓				Blepharoplasty	
					✓		✓		✓				Bicoronal Flap	
					✓				✓				Breast Augmentation	
					✓		✓		✓				Breast Reduction	
		✓			✓	✓	✓		✓				Canthoplasty/-pexy	
		✓			✓		✓		✓				Facial Lesions	
		✓			✓		✓		✓				Face Lift	
				✓	✓		✓		✓			✓	Forehead Corrugator	
		✓			✓		✓		✓				Otoplasty	
					✓		✓		✓				Platysma Band Notching	
		✓			✓		✓		✓				Rhinoplasty	
		✓			✓	✓	✓		✓				Transblepharoplasty Incisions	
					✓		✓		✓			✓	Scar Revision	
		✓			✓		✓		✓				Cleft Lip Revision	Reconstructive procedures
		✓				✓	✓						Cleft Palate	
					✓		✓		✓				Coronal Incisions	
					✓		✓		✓				Facial Incisions Trauma/Plating	
		✓			✓		✓		✓				Hand Surgery	
		✓			✓		✓		✓				Paediatric Plastic/Neuro	
		✓			✓	✓	✓		✓				Intra-oral Incisions	
		✓			✓		✓		✓				Otoplasty	
		✓		✓	✓		✓		✓				Skull Base	
		✓			✓		✓		✓				Foot Surgery	

Cotswold needle product codes														
PMS1696	PMS1695	PMS1693	PMS1692	PMS1656	PMS1654	PMS1653	PMS1651	PMS1650	PMS1614	PMS1613	PMS1612	PMS1605		
✓	✓	✓	✓	✓	✓		✓	✓	✓	✓	✓		Hip Endoprosthesis	Orthopaedic
✓	✓	✓	✓	✓	✓		✓	✓	✓	✓	✓		Knee Endoprosthesis	
		✓			✓		✓		✓				Frenectomy	Oral maxillofacial
		✓			✓	✓	✓		✓				Gingivectomy	
		✓			✓	✓	✓		✓				Intra-oral Incisions	
		✓			✓	✓	✓		✓				Palatal Donor Grafts	
					✓		✓		✓				Preauricular Incision for TMJ	
					✓		✓		✓				Submandibular Cutdown/Trauma	
					✓		✓		✓				Skin Incision for Hip Graft	
					✓		✓		✓				Coronal Incisions	
					✓		✓		✓				Laminectomy	Neurosurgery
					✓		✓		✓				Paediatric Incisions	
					✓		✓		✓				Spine/Rhizotomy	
		✓			✓	✓	✓		✓				Skull Base	
				✓	✓		✓		✓		✓		Neck Dissection	ENT procedures
					✓		✓		✓				Otoplasty	
					✓		✓		✓				Palatoplasty	
					✓		✓		✓				Rhinoplasty	
	✓			✓	✓				✓			✓	Tonsillectomy	
							✓						Tympanomeatal Flap	
					✓				✓				ENT Cosmetic	

Eurosurgical Ltd

Merrow Business Centre, Guildford, Surrey, GU4 7WA

+44 1483 456007 sales@eurosurgical.co.uk www.eurosurgical.co.uk